

## Lock N Load RG4200

Lock N Load™  
Exclusive Nova Technology Patent Pending

...to Fold and Go!



Lock N Load™  
Exclusive Nova Technology Patent Pending

Locked

Unlocked



Keeps your walker in a folded position for easy loading and unloading.

### Lock

1. Place the walker in the folded position and rest it on the rear wheels (see user manual).
2. Grasp the contoured pad at the base of the Lock N Load and push upward until the tip is inserted into the hinge opening.

### Unlock

1. Place the walker in the folded position and rest it on the rear wheels (see user manual).
2. Grasp the contoured pad at the base of the Lock N Load and push downward until the tip is removed from the hinge opening.
3. Open the walker by squeezing the hand brakes and pushing forward (see user manual).

The REDGUM Brand Lock N Load is designed keep seat walkers in the folded position when being moved or stored. It clips on to the seat walker for ease and convenience and is suitable for most brands of seat walker

Phone: 08 9248 4180

Fax: 08 9248 9452

E-mail: [orders@redgumbrand.com.au](mailto:orders@redgumbrand.com.au)